

v-gel[®] Guia de Tamaños

advanced

v-gel[®] Gato y Conejo

1. Determinar el peso ideal del paciente.
2. Seleccionar el v-gel[®] de mayor tamaño que se ajuste al peso ideal del paciente.
3. Si el v-gel[®] seleccionado no se ajusta bien a la faringe, elegir un tamaño menor.



CAT v-gel[®] Sizing Guide

Size	Ideal body weight of cat
C0	0.5 – 1kg (1 – 2lb)
C1	1 – 2kg (2.5 – 4.5lb)
C2	1.5 – 3.5kg (3 – 8lb)
C3	3 – 5kg (6.5 – 11lb)
C4	4.5 – 6kg (10 – 13lb)
C5	6kg+ (13lb+)



RABBIT v-gel[®] Sizing Guide

Size	Ideal body weight of rabbit
R1	0.6 – 1.5kg (1.3 – 3.5lb)
R2	1 – 2kg (2 – 4.5lb)
R3	1.8 – 3.5kg (4 – 8lb)
R4	2.5 – 4kg (5 – 9lb)
R5	3.5 – 5kg (8 – 11lb)
R6	4.5kg+ (10lb+)

v-gel[®] Perro

1. Seleccionar el tipo de paciente (Braqui, Meso, Doli)
2. Determinar el peso ideal del paciente.
3. De la columna de la derecha, elegir la referencia que se ajuste al tipo de paciente.



This table is intended to be a guide only. The right size v-gel[®] will be determined by the size and shape of a dog's pharynx, which varies depending on the breed, size and head and neck shape of the dog.

Head shape – Brachycephalic	Lean weight of dog	Head shape – Mesocephalic	Lean weight of dog	Head shape – Dolichocephalic	Lean weight of dog	Size/ Product code
	6 – 10kg 13 – 22lb		4 – 7kg 9 – 15lb		2 – 4kg 4.5 – 9lb	D3 = D20003
	9 – 15kg 20 – 33lb		6 – 10kg 13 – 22lb		4 – 7kg 9 – 15lb	D4 = D20004
14 – 20kg 30 – 45lb	9 – 15kg 20 – 33lb	6 – 10kg 13 – 22lb	D5 = D20005			
19 – 26kg 42 – 57lb	14 – 20kg 30 – 45lb	9 – 15kg 20 – 33lb	D6 = D20006			

Videos disponibles en www.docsinnovent.com/products

Jordi Gimeno
Pet Care & Consulting

di docsinnovent[®]

**v-gel[®] – the world's first
veterinary supraglottic airway device**