

v-gel[®] Guia de Tamaños

v-gel[®] Gato y Conejo

1. Determinar el peso ideal del paciente.
2. Seleccionar el v-gel[®] de mayor tamaño que se ajuste al peso ideal del paciente.
3. Si el v-gel[®] seleccionado no se ajusta bien a la faringe, elegir un tamaño menor.

 CAT v-gel[®] Sizing Guide		 RABBIT v-gel[®] Sizing Guide	
Size	Ideal body weight of cat	Size	Ideal body weight of rabbit
C0	0.5 – 1kg (1 – 2lb)	R1	0.6 – 1.5kg (1.3 – 3.5lb)
C1	1 – 2kg (2.5 – 4.5lb)	R2	1 – 2kg (2 – 4.5lb)
C2	1.5 – 3.5kg (3 – 8lb)	R3	1.8 – 3.5kg (4 – 8lb)
C3	3 – 5kg (6.5 – 11lb)	R4	2.5 – 4kg (5 – 9lb)
C4	4.5 – 6kg (10 – 13lb)	R5	3.5 – 5kg (8 – 11lb)
C5	6kg+ (13lb+)	R6	4.5kg+ (10lb+)

v-gel[®] Perro

1. Seleccionar el tipo de paciente (Braqui,Meso,Doli)
2. Determinar el peso ideal del paciente.
3. De la columna de la derecha, elegir la referencia que se ajuste al tipo de paciente.



This table is intended to be a guide only. The right size v-gel [®] will be determined by the size and shape of a dog's pharynx, which varies depending on the breed, size and head and neck shape of the dog.						
Head shape – Brachycephalic	Lean weight of dog	Head shape – Mesocephalic	Lean weight of dog	Head shape – Dolichocephalic	Lean weight of dog	Size/ Product code
 	6 – 10kg 13 – 22lb	 	4 – 7kg 9 – 15lb	 	2 – 4kg 4.5 – 9lb	D3 = D20003
	9 – 15kg 20 – 33lb		6 – 10kg 13 – 22lb		4 – 7kg 9 – 15lb	D4 = D20004
	14 – 20kg 30 – 45lb		9 – 15kg 20 – 33lb		6 – 10kg 13 – 22lb	D5 = D20005
	19 – 26kg 42 – 57lb		14 – 20kg 30 – 45lb		9 – 15kg 20 – 33lb	D6 = D20006

Videos disponibles en www.docsinnovent.com/products

Jordi Gimeno

Pet Care & Consulting

www.jordigimeno.com

 docsinnovent[®]

v-gel[®] – the world's first
veterinary supraglottic airway device